



OP-2UMN034

**NEW**

**MEDICNRG™**  
Precise Endo Technology

**Instructions for use  
Mini File Holder**



***Please note: the mini file holder  
is not supplied in a sterile state!***

Manufactured by MedicNRG Ltd., Israel  
Website: [www.medicnrg.com](http://www.medicnrg.com)  
Fax: +972-4-675-4278

**Congratulations!** You  
have purchased the most  
advanced file holder  
available for use with your  
apex locator.

*\* See other side for detailed photo.*



- Insert the mini file holder (2) into the socket at the tip of the cable (1).

**Please note:** the file holder should be inserted into the cable in one direction only - the slit opening for the file should be facing outwards, and not inside the socket.



- File sizes thicker than 0.2, should be slipped into the first notch. Inserting a large file into the second groove may widen the file holder and thin files may slip out. Thin files may be inserted into the second notch.

- Insert your file (3) (either NiTi or regular) into the slit opening at the tip of the mini-file holder (2) until it reaches the groove (4).

The file is now secure in the holder.

- The mini file holder should be appropriately sterilized after each use, and between patients.

**Please note:** The mini file holder is compatible with most endo handpieces